Distributed by

**BioMedical**  
B. Baumann Germany  
info@biomedical-baumann.com  
https://www.viscoplus.eu

**Bohus BioTech AB**  
Trädgårdsgatan 4  
SE-452 31 Strömstad, Sweden  
Phone: +46 526 14686  
E-mail: info@bohusbiootech.com  
http://www.bohusbiootech.com

# € 0483

## EN INSTRUCTIONS FOR USE

**Description**  
ViscoPlus® matrix (sodium hyaluronate) is a 2.5% viscoelastic, sterile and clear single dose solution for injection into the joint cavity. The highly purified sodium hyaluronate is dissolved in a physiological buffer at pH 6.8 -7.6.

**Composition per syringe**  
Each syringe with 3 ml (ViscoPlus® matrix) contains:

Sodium hyaluronate cross-linked (made by fermentation)	75 mg
Sodium chloride solution	q.s.
Physiological buffer	q.s.

**Intended use**  
Treatment of mild to moderate osteoarthritis in knee, hip, shoulder, ankle and temporomandibular joint. ViscoPlus® matrix can also give extra viscoelastic support to other joints with synovial capsule.

**Characteristics and mode of action**  
Hyaluronic acid is a native substance and hence very biocompatible. The most important properties of hyaluronic acid are protecting, lubricating and supporting of delicate cells and tissues. ViscoPlus® matrix, containing cross-linked sodium hyaluronate, free from avian protein, may ameliorate the viscoelastic properties of synovial fluid, thus the lubrication and shock-absorbing properties of synovial fluid are improved and the mechanic overload of the joints is reduced.

**Dosage and administration**  
ViscoPlus® matrix is injected into the affected joint only once. Several joints may be treated at the same time. Repeated treatment cycles may be administered as required at intervals of not less than 6 months. Take the pre-filled syringe out of the blister. Before use allow the preparation to equilibrate to room temperature for 20 minutes. Remove the tip cap. Mount the cannula by using the thumb and forefinger to hold firmly around both the glass syringe barrel and the luer-lock adapter. Grasp the cannula shield with the other hand. To facilitate proper assembly, both push and rotate firmly. Suitable cannula size (e. g. 18-22 G). Work aseptically to open the cannula and to remove the cap on the syringe.

**Contra-indications and precautions**

- Do not use in patients with ascertained hypersensitivity to the constituents in ViscoPlus® matrix.
- ViscoPlus® matrix must not be used when patients have septic arthritis. It is not indicated for use in other types of arthritis not related to osteoarthritis.
- ViscoPlus® matrix must only be injected by qualified medical personnel (orthopaedist, rheumatologist or equivalent). The dosage shall be prescribed by the physician, who will adjust the dosage specifically for the individual patient.

- The syringe is intended for single use only. The syringe shall be used immediately after the packaging has been opened.  
- Remove superfluous synovial fluid before injecting in the joint. ViscoPlus® matrix must be injected accurately into the joint cavity. Avoid injections into blood vessels and surrounding tissues.  
- General precautions for intra-articular injections shall be observed.  
- Do not use the product if the package or the single dose syringe is damaged.  
- Limited experience is available for the use of hyaluronic acid in children, pregnant and lactating women. Consult your doctor concerning ViscoPlus® matrix in these cases.  
- ViscoPlus® matrix may contain traces of BDDE, which in rare cases can cause tissue irritation.

**Side effects**  
Local secondary phenomena, such as pain, feeling of heat, redness of the skin (flush) and swelling may occur in a joint treated with ViscoPlus® matrix.

**Interactions with other products**  
Do not inject anaesthetics or other pharmaceutical preparations in the joint during the period of time the patient is treated with ViscoPlus® matrix.

**Please note**  
The solution for injection in the syringe is sterile. For storage and transportation the syringe is protected by a blister package in a carton. The outer surface of the pre-filled syringe is clean, but not sterile. Do not use syringe, if the pre-filled syringe or blister cover is damaged. Strict aseptic administration technique must be followed.  
Store between +2 °C and +25 °C. Protect from freezing and direct light.  
Shelf life: see labeling/markng on syringe and product carton.

To be used by a physician only.  
Store away from children!  
**Package size**  
1 Pre-filled syringe, containing 3 ml (ViscoPlus® matrix) of sterile solution for injection.

	Do not re-use		Steam sterilized		Do not use if package is damaged		Keep away from sunlight		Use-by date
	Consult instruction for use		Store between +2°C and +25°C		Batch code		Keep-dry		Manufacturer

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