ViscoPlus

CE 0483

	
Bohus BioTech AB Trädgårdsgatan 4 SE-452 31 Strömstad, Sweden Phone: +46 526 14686 E-mail: info@bohusbiootech.com http://www.bohusbiootech.com	

EN

INSTRUCTIONS FOR USE

Description
ViscoPlus® (sodium hyaluronate) is a 1,0 % viscoelastic, sterile and clear single dose solution for injection into the joint cavity. The highly purified sodium hyaluronate is dissolved in a physiological buffer at pH 6,8-7,6.

Composition per syringe
Each syringe with 2 ml (ViscoPlus®) contains:
Sodium hyaluronate (made by fermentation) 20 mg
Sodium chloride 16,6 mg
Disodium phosphate dihydrate 2,8 mg
Potassium dihydrogen phosphate 0,52 mg
Water for injections q.s.

Intended use
Treatment of mild to moderate osteoarthritis in the knee hip, shoulder, ankle and temporomandibular joint.
Characteristics and mode of action
Hyaluronic acid is a native substance and hence very biocompatible. The most important properties of hyaluronic acid are protecting, lubricating and supporting of delicate cells and tissues. ViscoPlus®, containing sodium hyaluronate, free of avian protein, may ameliorate the viscoelastic properties of synovial fluid, thus the lubrication and shock-absorbing properties of synovial fluid are improved and the mechanic overload of the joints is reduced.

Dosage and administration
ViscoPlus® is injected into the affected joint once a week for a minimum of 3 up to a total of 5 injections. Several joints may be treated at the same time. The effectiveness of a single treatment cycle of less than 3 injections has not been established. Repeated treatment cycles may be administered as required at intervals of not less than 6 months.
Take the pre-filled syringe out of the blister. Remove the cap by twisting and immediately attach a suitable needle (e.g. 19 or 21 G). Work aseptically to open the needle and to remove the cap on the syringe.

Contra-indications and precautions
- Do not use in patients with ascertained hypersensitivity to the constituents in ViscoPlus®.
- ViscoPlus® must not be used when patients have septic arthritis. It is not indicated for use in other types of arthritis not related to osteoarthritis.
- ViscoPlus® must only be injected by qualified medical personnel (orthopaedist, rheumatologist or equivalent). The dosage shall be prescribed by the physician, who will adjust the dosage specifically for the individual patient.
- The syringe is intended for single use only. The syringe shall be used immediately after the packaging has been opened.
- Remove superfluous synovial fluid before injecting in the joint. ViscoPlus® must be injected accurately into the joint cavity. Avoid injections into blood vessels and surrounding tissues.
- General precautions for intra-articular injections shall be observed.

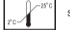
- Do not use the product if the package or the single dose syringe is damaged.

- Limited experience is available for the use of hyaluronic acid in children, pregnant and lactating women. Consult your doctor concerning ViscoPlus® in these cases.

Side effects
Local secondary phenomena, such as pain, feeling of heat, redness of the skin (flush) and swelling may occur in a joint treated with ViscoPlus®.

Interactions with other products
Do not inject anaesthetics or other pharmaceutical preparations in the joint during the period of time the patient is treated with ViscoPlus®.

Please note
The solution for injection in the syringe is sterile. For storage and transportation the syringe is protected by a blister package in a carton. The outer surface of the pre-filled syringe is clean, but not sterile. Do not use syringe, if the pre-filled syringe or blister cover is damaged. Strict aseptic administration technique must be followed.
Store between +2 °C and +25 °C. Protect from freezing and direct light.
Shelf life: see labeling/markng on syringe and product carton.
To be used by a physician only.
Store away from children!
Package size
1 Pre-filled syringe, containing 2 ml (ViscoPlus®) of sterile solution for injection.

	Do not re-use		Steam sterilized		Do not use if package is damaged		Keep away from sunlight		Use-by date
	Consult instruction for use		Store between +2°C and +25°C		Batch code		Keep dry		Manufacturer

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